QC:

Name:

Signature::

Product identification:

Distributor:

Date of sale (day, month, year):

Name, stamp and signature of the seller:

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IFU-scope code: „MULTI 1-EU“:

INFORMATION FOR USE (ENG)

=> medical device: rehab-stroller (buggy)

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
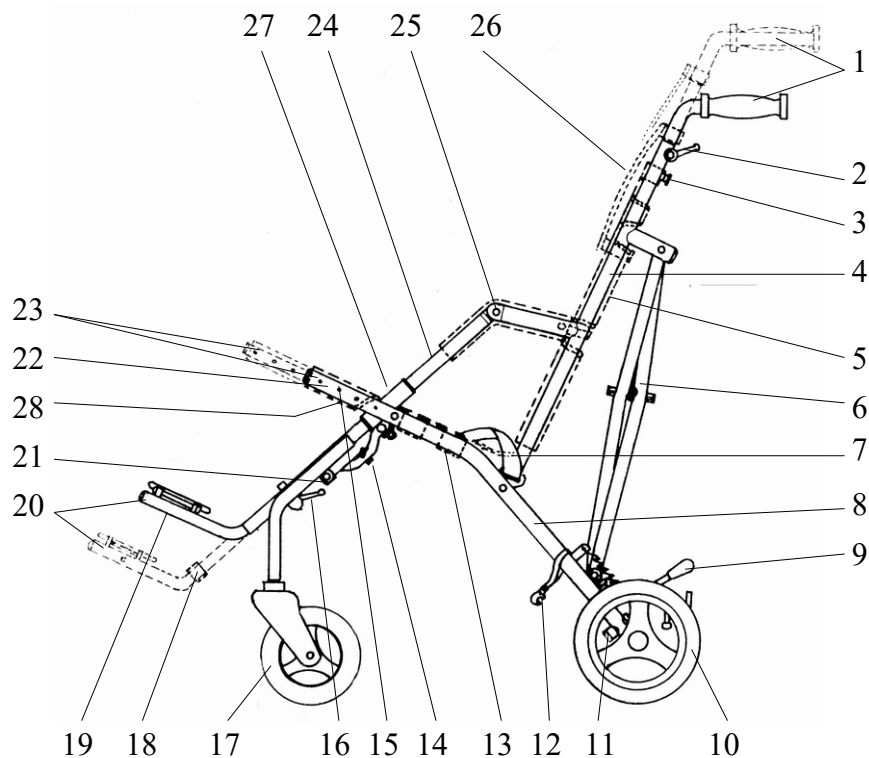
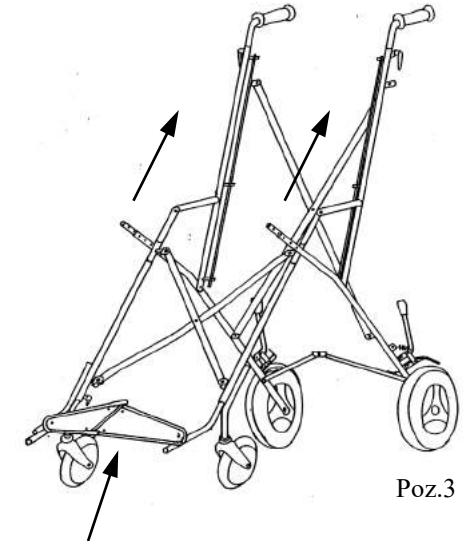
SCHEME

Dear customers,

Rehab-Stroller CORZO are designed for short stay of an occupant. With the design and especially unique construction are ranked among the best from their category. Prior the first usage of the buggy, please, read carefully this user guide. In case you are not fully sure about the proper usage of the product after reading of the user guide feel free to contact your local dealer. Retain the user guide for future reference.

**WARNINGS AND SAFETY NOTES:**

① **CAUTION! Any load attached to the handle (bag, ...) affects the stability of the stroller!**

.....maximum load,

.....weight of the product

-WARNING! Keep fingers and cover out of any moving parts when operating with the stroller

-WARNING! Never leave your child unattended! Even if the belts and brakes are in use.

-WARNING! Check before use that all locking devices are properly locked and all safety equipments are properly used.

-WARNING! Check before use that seat unit is properly locked in the chassis interface.

-WARNING! The seat unit is not suitable for a child younger 6 months.

-WARNING! This product must not be used while jogging or in-line skating.

-WARNING! The seat unit must not be used in motor vehicles as the car seat.

-ATTENTION! Read this manual carefully before the first use of the stroller, as well as keep it for further reference. Safety of a child might be in danger in case you will not follow the instructions and safety notices mentioned I the user guide.

- The attendant must be familiar with the instructions of this manual.
- Professional medical advice regarding the suitability of this product must be sought before use.
- Always make sure, before departure, the child is comfortable, and that clothes and rugs etc, are tucked in, so they do not fall in the wheels.
- The stroller MUST NOT be used as a seat for a child while traveling in motor vehicle or bus.
- Ensure that child is clear of moving parts before making adjustments, to avoid any injury.
- When angle-positioning and adjusting the seat and backrest act accordingly to avoid sudden uncontrolled movements.
- Change of the seat direction must be done without the occupant and with empty seat unit.
- Do not allow the child to climb unassisted into and out from the stroller.
- Always apply the parking brakes, when getting in and out the stroller.
- Ensure yourself that the fixing accessories, safety bar and other equipment is fitted and adjusted correctly.
- Do not stand on the footrest.
-
- The brakes must be engaged at all times when the stroller is not moving.
- Getting over the curb practice firstly with an empty stroller.
- The stroller is not designed to be driven down the stairs.
- The attendant must take precautions against the fire risk. In buildings, check the location of fire exits and ensure procedures are understood.
- Do not let children play with or hang on the stroller.
- During the drive keep full attention, to avoid any dangerous situation or collisions.
- Do not overload the stroller. Overloading of a chassis, seat, bag or basket causes reduction of stability of the stroller.
- Make the full attention, when manipulate with the stroller, not to injure the child or yourself.
- If it is possible, always use drive-up ramps or lift to overcome any access restrictions.
- Always lift the stroller in two persons
-
- Never transport the stroller when you and/or assistance are not sure if you can transport the stroller with the child safely.
- When you drive the stroller on the road, follow the road traffic regulations.
- Do not use the stroller for any other purposes then it is designed for.

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- The stroller is designed to carry only one child.

- Do not use any other than original PATRON accessories designed for this product.

- When the stroller is folded and unfolded, the user guide must be followed carefully, to avoid any damage on the construction what could affect the product stability.

- In case of apparent defect, which limits product's safe using (e.g. damage of chassis, part falls away or other optically visible defects, cracks etc.) it is necessary to stop using the product immediately and contact your local official distributor or service centre.

- Company PATRON does not bear any responsibility for the damages caused by using other than original PATRON accessories supplied by this company.

- Company PATRON does not bear any responsibility for the damages and accidents caused by inappropriate use of this product.

INTRUCTIONS FOR USE:**Opening the buggy:**

stand buggy on wheels, unlock storage clip, grasp one handle and push down on the opposite seat tube. Press down firmly on seat tubes to ensure the buggy is completely opened. Use foot to lock the rear brace into a straight position. Wrap the armrest padding around the armrests. Mount the footrest on the buggy. (Lock the seat brace - size - 38)

Footrest:

install - put on the floor almost decomposed braked stroller and release both safety levers. Take the footrest, press spring button placed at the rear top of the tube, unlock the locking pin and pull both tubes into the both footrest attachments until they are locked in the lowest position by the rear spring button. height adjustment - release both safety levers, pull both spring buttons of the footrest and adjust it into desired position. Tighten both safety levers to secure the footrest in position.

uninstall - put on the floor almost decomposed braked stroller. Place the footrest into its lowest position, press the rear spring buttons, unlock the locking pin and pull out the footrests both side tubes at the same time.

Seat depth adjustment:

the seat should support the tights on their whole length. To change the seat depth undo the velcro panel connecting the seat and backrest and partially fold the buggy to relieve fabric tension. Grasp end of seat tube, press seat spring button and move seat tube until spring button relocates into the desired hole. Repeat for other side of

seat. Fully open the buggy and fasten the velcro panel connecting the seat and backrest.

Backrest and handles height adjustment:

undo the velcro straps of the backrest extension and partially fold the buggy to relieve fabric tension. Grasp the handle, release handle safety lever, pull out the handle quick release, move the handle until it relocates into desired position. Release the handle quick release by moving the handle. Tighten the handle safety lever. Repeat for other handle and fully open the buggy. Fasten all velcro straps of the backrest extension.

Brakes:

To engage the brake, press brake lever pad on both sides fully till the end position. To disengage the parking brake, lift both brake lever pads.

Five-point belt:

release all plastic quick fasteners (fork type), place the child into the buggy and adjust of desired lengths the belt. Choose the hole in backrest to adjust the proper position of the shoulders straps. The hole should be level with or higher than the top of the shoulders. The shoulders straps must be mounted on backrest and backrest extension at once when possible.

Getting into the Buggy:

ensure the parking brakes are on, to make access easier, the footrest can be taken off; if possible, place the back of the buggy against a wall or immovable object (alternatively, another helper may hold the buggy steady); place child carefully into the buggy, fasten safety belt and fit the footrest. The child must always feel comfortable.

Getting out the Buggy:

ensure the parking brakes are on; if possible, place the

back of the buggy against a wall or immovable object (alternatively, another helper may hold the pushchair steady); unfasten safety belt and lift the child carefully.

Negotiating the Kerb or Single Step:

down - take the rear wheels to the edge of the kerb, hold the handles firmly and tip the buggy slightly backwards. Balance on the rear wheels, lower the buggy down the kerb on its rear wheels. Never tip the buggy forward at a kerb.

up - take the rear wheels to the kerb. With a firm grip on the handles, tip the buggy backwards and place the rear wheels on top of the kerb. Position the front wheels at the kerb bottom and lift to the upper level.

Negotiating Steps:

- make use of ramps and lifts when they are available
- where possible steps should be avoided, if there is no alternative to negotiating steps then ensure at least one helper is available to give assistance

do not attempt to take the buggy up or down a flight of steps unless you and your helper are sure you have control of the buggy and can take the weight .

- of its occupant
- during this operation, the attendant and helper must be well balanced and have secure grip on the buggy
- the attendants hands must be inside the handle straps at all time
- take rests in regular intervals
- do not lift the buggy with footrest

going down steps - position the buggy squarely with the front wheels on the top step, tilt the handle backward and push the buggy forward until the rear wheels are at the top step; the helper stands in front, steadies and lowers the buggy by gripping the front frame; take the weight and lower the rear wheels down the step, continue step by step to the bottom. (picture no. 1)

going up steps - position the buggy squarely with the rear wheels against the bottom step, tilt the buggy backward and raise to the next step, the helper stands in front, steadies and lifts the buggy by gripping the front frame, continue step by step to the top (picture no. 2)

Folding the buggy:

unfasten the dry zipper of armrest upholstery, unlock rear brace by pushing up on centre of brace. (Unlock the seat brace - size 38) Grasp one handle and pull up the seat tube with other hand. Make sure to cover the backrest is curved toward the stroller to prevent damage to the spacer folding mechanism.

Lock the storage clip. Is not necessary to uninstall the footrest for folding the buggy. (Picture no.3)

Upholstery:

the seat, seat reinforcement, backrest and backrest extension are not detachable. You could clean them by warm water and mild detergent.

Armrests:

undo the velcro and button to uninstall the armrests upholstery.

Wash by hands on max. 40°C with mild detergent.

Seat reinforcement:

you could increase or decrease the seat rigidity by adjustment of the seat reinforcement. Lengthen the belts of seat reinforcement to decrease the rigidity. Shorten the

belts of seat reinforcement to increase the rigidity. Never change the seat width by adjusting of the seat rigidity.

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MAINTENANCE

- your buggy has been designed to a high standard and with correct use and maintenance will give many years of trouble free service
- regularly inspect locking devices, catches, footrest, joints fixings to ensure they are secure and in full working order
- brakes, wheels, tires are subjects to wear and should be replaced if necessary by a repairer approved by the buggy supplier
- we recommend that the buggy is checked by an approved repairer every 12 month.
- We recommend that after 24 months visit a service department.
- if the buggy is damaged or in need of repair, the approved repairer will be able to advise on what is required.
- alternatively, PATRON Bohemia a.s. Customer department is available for product information.
- only PATRON Bohemia a.s. replacement parts should be used on this buggy.
- should parts of your buggy become stiff or difficult to operate (or not later than every 6 month), ensure that dirt is not the cause, apply a light application of a spray lubricant e. g. WD 40 (do not use oil or grease). Ensure that the lubricant penetrates between the components but do not over spray as this may encourage the accumulation of dirt. Wipe off any access spray. Regular lubrication of moving parts will extend the life of your buggy and take opening and folding easier.
- make sure that you are familiar with the operation of the buggy. It should always open and fold easily. If it does not, do not force the mechanism - stop and read the instructions.
- wheels, plastic and metal parts may be sponged clean with warm water and mild detergent.
- the seat fabric can be washed by hands (max. 30°C).
- never clean with abrasive, ammonia based, bleach based or spirit type cleaners.
- after exposure to damp conditions, dry off with a soft cloth and leave fully open in warm environment.
- store your buggy in a dry, safe place.
- Check and tighten the screw connections regularly.
- If there are used control cables (brakes, telescopes, tilt), check it regularly and, if necessary, set up and oil them.

PATRON Bohemia a.s. reserves the right to make design changes to any of their products without prior notice as a part of their continuous development program

Warranty conditions:

1. On this product is applied 24 months warranty from the date of sale. The warranty is applied for material and manufacturing defects.
2. The warranty has to be applied in the business organi-

zation, where the product was bought, or in authorized service repair shop.

3. **Properly filled Certificate of Warranty and a copy of acquiring document have to be advanced when applying the warranty. Do not pull down the identification card. It has to be readable and undamaged for the whole warranty period.**
4. Any defect that has occurred must be repaired immediately after finding it. the user is fully responsible for defects resulting from the continued use of the product without the original defect being repaired.
5. The product has to be used entirely to the purpose, which it is determined for, the user has to follow the instructions mentioned in the instructions for use. Producer / distributor is not responsible for defects caused by incompetent handling in conflict with the instructions for use.
6. The product has to be used, treated, maintained and stored properly according to directions mentioned in the instructions for use.
7. The warranty proceedings can be initiated, if the product under claim has been sent complete in original package, carefully packed and secured against harm. The owner is obliged to advance the product under claim properly cleaned up, hygienically flawless due to common hygienic principles. The owner cannot ask the seller to do basic maintenance or cleaning of the product during claim proceedings.
8. Every defect has to be claimed immediately
9. The warranty is forfeit if the defects were caused by forbidden overloading, accident, violence or incompetent intervention into the product or incompetent handling.
10. Warranty period cannot be interchanged with using period, i.e. period, for which, when used and treated properly including maintenance, the product, in respect of its characteristics, given purpose and difference in construction, can be used properly.
11. The engineering solution does not guarantee the ideal direction trails.

NOTES:

5. Changes on all the products of the company PATRON Bohemia a.s. are reserved.

SCHEME DESCRIPTION

1 handle	15 seat safety pin	16 footrest adjusting lever
2 handle adjusting lever	17 front wheel	
3 handle adjusting safety pin	18 calf rest	
4 Backrest frame	19 footrest safety	
5 Backrest	20 footrest	
6 cross-frame	21 arrestment device spring	
7 safety belt	22 seat	
8 seat frame	23 extension seat adjustment	
9 parking brake	24 side frame	
10 rear wheel	25 side frame cover	
11 back safety crossbar	26 backrest height extension	
12 transport safety-catch	27 frame	
13 seat reinforcement	28 seat clip reinforcement	
14 seat crossbar		

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